IFU LANGUAGE INDEX

				PAGE
BLG		Български	Bulgarian	2
HRV		Hrvatski	Croatian	3
CZE		Čeština	Czech	4
DAN		Danish	Dansk	5
EST		Eesti keel	Estonian	6
FIN		Suomen kieli	Finnish	7
FRA		Français	French	8
DEU		Deutsch	German	9
GRE		Ελληνικά	Greek	10
HUN		Magyar	Hungarian	11
GLE		Gaeilge	Irish	12
ITA		Italiano	Italian	13
LAV		Latviešu	Latvian	14
LIT		Lietuvių	Lithuanian	15
MLT		Malti	Maltese	16
DUT		Nederlands	Dutch	17
POL		Polski	Polish	18
POR		Português	Portuguese	19
RUM		Română	Romanian	20
SLO		Slovenčina	Slovak	21
SLV		Slovenščina	Slovenian	22
SPA		Español	Spanish	23
SWE		Svenska	Swedish	24
ENG		English	English	25

Rev. G-05
2025-07-01

Mesa Italia S.r.l. via dell'Artigianato, 35/37, 25039 Travagliato (BS)
ITALY Tel. +39 0306863251
info@mesaitalia.it www.mesaitalia.it

MAGNUM SALDATURA Co **CE 0425**

LOT00000

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MAGNUM SALDATURA Co **CE 0425**

LOT00000

Cobalt based dental alloy for welding
According to ISO 9333:2006

The product is intended for dental laboratories for the creation of products that must be rigid and resistant, single crowns, bridges and superstructures on implants. The alloy MAGNUM SALDATURA Co was studied to weld cobalt based alloys. It's free of nickel and completely free of toxic elements like beryllium, gallium and indium. MAGNUM SALDATURA Co is a product intended to be used by qualified technician.

Intended purpose

Welding of metal prostheses.

Intended patient population

This alloy is intended to be used for oral rehabilitation in edentulous or partially edentulous patients who suffer from loss of hard dental tissue. There are no limitations related to age, sex, clinical condition or other conditions, other than those possibly determined case by case by the professional user.

Intended users

The product is intended for professionals of the dental sector, in particular: prosthodontic laboratories, dental technicians, dentists. These professional figures shall have a high degree of specialisation and are perfectly aware of the destination of the product and the correct method of use for the realization of a final product and elaborate different methods to reach the best shape of the final prosthesis. Contact and/or manipulation by other people is not envisaged.

Instruction for use

Open flame welding

After having cleaned from the oxid the surfaces to weld, cover them with flux. Heat slowly the parts to be soldered with the flame and lay a homogeneous welding material.

Re-use

The device can be reused until the available material is exhausted.

Waste disposal

The processing scrap must be disposed of as special waste in accordance with the EC directives 2008/98/CEE on waste, and 94/62/CEE on packaging and packaging waste and in compliance with national legislation in force on the subject.

Safety instructions

- Cobalt-Chrome-based alloys can seldom cause dermatitis on sensitive subjects. A Patch-Test is thus advisable.
- Metal dusts and smoke are dangerous for health. Use exhaust fans while casting and polishing.
- The alloy contains Cobalt, element classified as CMR (CAS Number: 7440-48-4).
- Before prosthesis application verify if other metal elements are in patient's oral cavity. Coexistence of different metals can cause a "pile" effect.
- We identify every batch with a number. We recommend to write it down in patient's file to allow its complete traceability.
- This product does not need any special preservation precautions. MESA keeps batch traceability for 15 years.
- The methods of preserving the products made with MAGNUM SALDATURA Co are under the responsibility of professional operators (dental technicians). This product must be stored, transported and used at a temperature between -30°C and +50°C to ensure the integrity of the packaging and the instructions contained therein.
- MESA recommends that the patient be made aware of the possibility for dental alloys to affect MRI results.
- The product is sold non-sterile.

Users shall report any major accident occurred in relation to the use of this medical device to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

A summary of safety and clinical performance (SSCP) is available on Eudamed.

Symbol	Description	Symbol	Description
	Batch number		Date of manufacture
	Consult instructions for use		
	Manufacturer		CE Marking
	Weight estimate		Non-sterile
	Medical device		Catalogue number
	Temperature limitation		Contains hazardous substances
	Product UDI Code		

Percentage chemical composition (m/m)			
Co	62	Mo	4
Cr	29	Si	3.5
Others: C, Mn, Fe			

Physical and mechanical properties	
Density	8.2 g/cm ³
Solidus-liquidus temperature	1071-1260°C
Melting point	1310°C
Colour	White
Thermal expansion coefficient 25-500°C	$15.5 \cdot 10^{-6} \text{ K}^{-1}$
Thermal expansion coefficient 25-600°C	$15.8 \cdot 10^{-6} \text{ K}^{-1}$
Cytotoxicity test according to ISO 10993-5	Passed