

Instructions for use

Product name:Model Resin

Model: See label.

Composition: This product is composed of acrylic resin, acrylate monomer, photoinitiator, and additives.

Intended purpose: Used for making dental Model.

Product performance:

The main performance indicators are as follows

Appearance	The product is a homogeneous liquid, free of suspended matter, granular sediments and impurities.
Viscosity	The viscosity of the product at room temperature is 100~1500mpa • s.
Density	The density of the product is 1.10±0.5g/cm ³ .

Precautions:

- ① The user of this product must be trained by a qualified professional, while ensuring that the content of this instruction manual is fully understood before use;
- ② Please check the product carefully before use, packaging damage prohibited;
- ③ Different types of resins should not be mixed;
- ④ Please shake well before each addition, and avoid direct exposure to ultraviolet rays during use;
- ⑤ After reaching the expiration date of use, dispose of the product according to local regulations;
- ⑥ Hold the product gently and put it down lightly to avoid damage to the product due to falling, and do not use the product if it has been damaged after falling.

Warning and informative instructions:

After opening, keep away from light and keep out of the reach of children. In case of inadvertent contact with eyes or skin, flush immediately with plenty of water. Seek medical attention if necessary.

Instructions for use:

- ① The user should ensure that he/she is wearing overalls and gloves before use.
- ② Shake the resin well before adding and pour the Surgical Guide Resin into the resin tank.
- ③ Print according to the operating procedures of the 3D printing equipment manufacturer.
- ④ Use ethanol to clean the model after printing.

Package: This product is packaged in a black bottle.








Storage condition: This product should be stored in a dry, sealed environment protected from light.

Shipping Condition: In the process of transportation, it is necessary to prevent severe extrusion and vibration, prevent snow and rain from being soaked, and must not be mixed with corrosive substances.

Date of manufacture: See label.

Validity period: 1 year.

Description of label symbols:

	Date of manufacture		Batch code		Consult instructions for use or consult electronic instructions for use
	Caution		Do not re-use		Manufacturer
	CE mark				

Manufacturer's name: Chengdu Besmile Medical Technology Co., Ltd.

Manufacturer's address: No.9, Sec.2, Shengwucheng North Rd., Chengdu Tianfu International Bio-Town, Shuangliu District, Chengdu, Sichuan 610200, P.R.China.

Contact information: 028- 68907833